DILUENT-2

Buffer for HumaStar

IVD

Package Size

REF	Format	Unit/Size
16663/11	Buffer	4 x 20 ml

Intended Purpose

The Diluent-2 is a supporting buffer for manually or on-board dilution of human serum, plasma or urine samples and for preparing reagent blanks. For use with automated HumaStar systems only. For laboratory professional use only.

Contents

DIL-2, pH 5.0 -7.0	
DILUENT-2	
Preservative (Proclin 300)	9 ml/l

Suitable Human Devices

HUMAN System	REF
HumaStar 300SR	16930
HumaStar 600	16660

Preparation

DIL-2 is ready for use and can directly be applied on the analyser.

For further instructions, please refer to the user manual of the respective analyser.

Storage / Stability

Original sealed DIL-2 is stable up to the stated expiry date when stored at 2-25 °C. After first opening the stability is as follows:

Buffer	2-25 °C
DIL-2	30 days

Do not freeze. Contamination must be avoided.

Limitations

Diluent should not be used in combination with sodium or chloride reagents or with assays based on sodium or chloride reaction principles.

Safety Notes

All patient specimens, calibrators and controls should be handled as potentially infectious.

P234 Keep only in original container.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P262 Do not get in eyes, on skin, or on clothing.

P281 Use personal protective equipment as required.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P401 Store in accordance with local/regional/national/international regulations.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

For users in the European Union only: Please report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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