+++ Completely revised  +++ Please read carefully! +++

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**IVD**

### Package Size

| REF   | Format            | Unit/Size  |
|-------|-------------------|------------|
| 18974 | Cleaning Solution | 12 x 30 ml |

### Intended Purpose

The Special Wash Solution is a concentrate for the manual preparation of HumaStar cleaning solution. It is an accessory to ensure failure-free working of HumaStar analyzers by removal of sample or reagent residues. For laboratory professional use only.

### Contents

|                              |                  |
|------------------------------|------------------|
| [SWA] (concentrated), pH >13 |                  |
| NaOH                         | 2 mol/l (5-10 %) |

### Suitable Human Devices

| HUMAN System   | REF   |
|----------------|-------|
| HumaStar 100   | 16890 |
| HumaStar 200   | 16895 |
| HumaStar 300SR | 16930 |

### Preparation

Concentrated [SWA] needs to be diluted with deionised and filtered water (no particles) before being applied on the analyser.

Add deionised water to an appropriate preparation tank that can be closed. Carefully open a bottle of [SWA] concentrate and pipette the amount required into the preparation tank:

|                               | Special Wash Tank  | Special<br>Cuvette and Needle<br>Wash procedures                                     |
|-------------------------------|--|--|
| <b>Dilution ratio</b>         | approx. 1:33<br>For 1 l wash solution add 30 ml [SWA] (one bottle) to 970 ml deionised water | 3:5<br>For 50 ml wash solution add 30 ml [SWA] (one bottle) to 20 ml deionised water |
| <b>Final C<sub>NaOH</sub></b> | 60 mmol/l  | 1.2 mol/l  |
| <b>Tank for intended use</b>  | Special Wash Solution tank, green mark (2 l)   | Reagent bottle (50 ml)   |

Close the tank and shake it slightly to mix the solution. Avoid the creation of foam. Pour the prepared mix in the tank for the intended use. Use a clean uncontaminated funnel. If necessary, wait until bubbles and foam disappear.

For further instructions, please refer to the user manual of the respective analyser.

### Storage / Stability

Original sealed [SWA] concentrate is stable up to the stated expiry date when stored at 2-25 °C. After first opening/ dilution the stability is as follows:

|  |                              |
|--|------------------------------|
| Cleaning Solution                              | 2-25 °C                      |
| [SWA] concentrate (C <sub>NaOH</sub> =2 mol/l) | up to the stated expiry date |
| [SWA] diluted (C <sub>NaOH</sub> =60 mmol/l)   | 1 week                       |
| [SWA] diluted (C <sub>NaOH</sub> =1.2 mol/l)   | 6 months                     |

Avoid direct exposure to sunlight. Use only disposable pipettes to avoid contamination.

### Safety Notes

All patient specimens, calibrators and controls should be handled as potentially infectious.

[SWA] Danger

H290 May be corrosive to metals.

H314 Causes severe skin burns and eye damage.

[SWA]

P234 Keep only in original container.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P262 Do not get in eyes, on skin, or on clothing.

P281 Use personal protective equipment as required.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P401 Store in accordance with local/regional/national/international regulations.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

For users in the European Union only: Please report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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